

## Memo: Nova 2.0 — Urine Pregnancy Manual Entry Change

This communication is from:

Jieli Li, M.D., Ph.D.

Director, Clinical Pathology Services / Co-Director, Clinical Chemistry and Toxicology Laboratory

On behalf of:

Laboratory Compliance and Point of Care Testing Department

Applicable to: All Point of Care Testing Personnel

### OVERVIEW

The Nova 2.0 (Next Generation Meter) urine pregnancy manual entry screen differs from the legacy meters.



### WHAT'S DIFFERENT ON NOVA 2.0?

- Legacy meters displayed options **side-by-side** (left/right).
- Nova 2.0 displays options **stacked** (up/down).

### ACTION NEEDED


**Control line verification (manual entry):** Verify the control line is present and select **Present** on the meter.

If **Absent** is selected, the test is **invalid** and the result **will not post** to the patient chart. Follow-up action is needed.

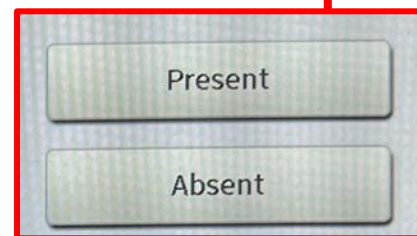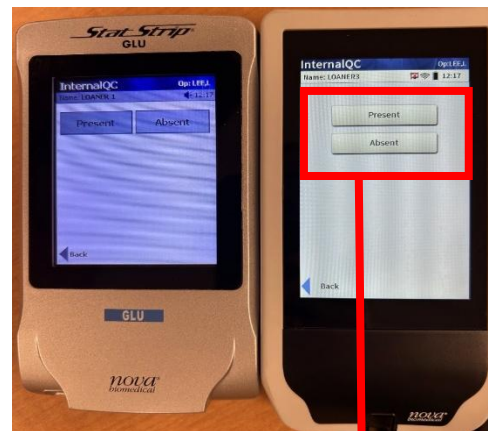

### QUICK STEPS (NOVA 2.0):

1. Start urine pregnancy manual entry on Nova 2.0 and follow on-screen prompts.
2. Confirm the control line is present; using the **up/down** layout, select **Present**.
3. Submit the entry. If the control line is not present, select **Absent** (invalid) and repeat testing per local procedure.

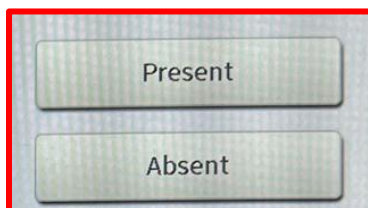

Present = Control line is present. Testing is valid.

Absent = Control line NOT present. Test is invalid. Results will NOT post. Repeat testing per procedure.

Please reach out to the POC Team ([path.glucose@osumc.edu](mailto:path.glucose@osumc.edu)) if you have questions or need clarification.